HICO-DECUBIMAT 375

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HICO-DECUBIMAT 375

HICO-DECUBIMAT 375, the efficient alternating pressure system, designed for high requirements in the care of decubitus patients, thus assisting the nursing personnel.

- 2-chamber big cell mattress with CPR-valve and static head cells
- high lying comfort due to air permeable and nonskid cover
- optimum support pressure relief for patients of 40 - 160 kgs
- hollow bedding for direct pressure relief possible
- transport mode and static soft lying
- transportation mode and static soft lying
- for prophylaxis and therapy up to stage II SEILER scale
- for pain relief
- easy operation and very low noise level

HICO-DECUBIMAT 375 contains a **2-chamber big cell mattress** and every single cell can be replaced. The material is kind to the skin and pervious to air. In the case of emergency a **CPR-valve** (fig. 1) provides for quick deflation of the cells.

Fig. 1: Mattres with quick deflation

The whole system, aggregate and mattress, is easy to operate. The **big cell mattress** with **2-chamber system** provides a good surface pressure relief with the partially changing zero pressure. For a better lying comfort the three head cells are static.

HICO-DECUBIMAT 375 provides a **transportation mode** (fig. 2). This fills all cells of the mattress up to the right pressure and thus guarantees a **soft lying**. After switching off the device, the pressure in the mattress remains constant. The patient can be **transported** with his bed, mattress and unit.

Fig 2: Operation displays

By microprocessor control the aggregate can be operated intuitively without problems.

Operations

The user is informed about the state of mattress and unit from the displays at any time.

A common **bacteria filter** (fig. 3) provides for a long lasting service life. It protects the system from contamination and can be replaced at any time. An important contribution to **operational safety** and **hygienics**.

Fig. 3: Replaceable bacteria filter

Indication

The **HICO-DECUBIMAT 375** serves the prophylaxis of low to moderate risk according to NORTON and the therapy of decubitus stages 1-2 according to SEILER in the case of bedridden patients. Furthermore the system offers a good lying comfort for patients from 40 to 160 kgs due to its 2-chamber big cell mattress.

Technical Data:

Unit:

Voltage:	230 V AC 50/60 Hz
Nominal consumption:	30 W
Current consumption (24h):	approx. 0,35 KWh
Dimension of unit:	400 x 350 x 150 mm
Weight:	approx. 6 kgs
Cycle time:	abt. 6 min.
Sound level:	approx. 30 dB (A)
Patient weight:	40 - 160 kgs
Risk class (93/42/EEC):	I
Testing base:	EN 60601-1

Mattress:

Design:	2-chamber system with 3 static head cells and CPR-valve
Cover:	kind to skin, pervious to air, waterproof, nonskid, ductile washable by machine
Weight:	approx. 4,7 kgs
Size: (in operating condition)	200 x 90 x 21 cms approx.

The right for alterations is reserved.

REF:

HICO-DECUBIMAT 375 cpl.: (unit, mattress with cover, bacteria filter and manual)	35 00 00
---	----------

Warrenty:

The **HICO-DECUBIMAT 375** corresponds to the EC-guideline 93/42/EEC and bears the CE-mark. The unit is produced and tested by HIRTZ & CO. according to ISO 9001, EN 46001 and EN 60601-1. For **HICO-DECUBIMAT 375** we grant a warrenty of 24 months.